Cleaning and care of vinyl upholstery

Instructions for use

RUCK® Cleaning and care of vinyl upholstery cleans and cares for all seat covers made from artificial leather and is highly effective in maintaining the value of treatment chairs and other artificial leather covers. You can apply RUCK® Cleaning and care of vinyl upholstery with the special sponge supplied in the set to rapidly and thoroughly remove stubborn grime, stains, and discoloration, caused for example by non-fast dyestuffs in clothing. RUCK® Cleaning and care of vinyl upholstery cleans and cares intensively without leaving any residues; it provides a silky gloss finish.

Composition:

RUCK® Cleaning and care of vinyl upholstery is based on a mixture of special tensides, silicone compounds and care components derived from avocado oil.

Application:

Use RUCK® Cleaning and care of vinyl upholstery undiluted. Spray RUCK® Cleaning and care of vinyl upholstery onto the surfaces to be cleaned and then immediately wipe off with a cloth. For intensive cleaning rub RUCK® Cleaning and care of vinyl upholstery on with the special sponge, which achieves an even more thorough and deeper cleaning effect. Subsequently, wipe off the excess RUCK® Cleaning and care of vinyl upholstery with a dry cloth. Use RUCK® Cleaning and care of vinyl upholstery daily if required.

Notes:

RUCK® Cleaning and care of vinyl upholstery can be used without any problems on all artificial leather surfaces. The material tolerability of other leather surfaces should be checked on a concealed surface. The special sponges should not be used on leather. Do not mix RUCK® Cleaning and care of vinyl upholstery with other products. Changes in colour and odour of RUCK® Cleaning and care of vinyl upholstery during storage have no impact on efficacy. For professional use only. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

RUCK. Cleaning and care of vinyl upholstery

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Copyright

© 2023 HELLMUT RUCK GmbH. All rights reserved. This document must not be copied, edited, reproduced or translated without the prior written permission of Hellmut Ruck GmbH.

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**orochemie GmbH + Co. KG | Max-Planck-Straße 27
Distribution by: HELLMUT RUCK GmbH | Daimlerstraße 23 | DE – 75305 Neuenbürg