Contents

Bulgarian.....	2
Croatian.....	4
Czech.....	6
Danish.....	8
Dutch.....	10
English.....	12
Estonian.....	14
Finnish.....	16
French.....	18
German.....	20
Greek.....	22
Hungarian.....	24
Irish.....	26
Italian.....	28
Latvian.....	30
Lithuanian.....	32
Maltese.....	34
Polish.....	36
Portuguese.....	38
Romanian.....	40
Slovak.....	42
Slovenian.....	44
Spanish.....	46
Swedish.....	48

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Rev.: 03

General information

- The instruments are always delivered in a non-sterile condition and must be processed using suitable methods before use. Failure to follow the reprocessing instructions below can result in the transfer of pathogens.
- Disposable products, which are identified by the symbol (☒), may be processed only once before they are used for the first time and may not be reused.
- Keep the packaging – also during the active use phase of the instrument – in order to ensure that the instrument can be traced if necessary.
- Check instruments for corrosion, cutting edge/diamond coating condition, and shape damage before each use. Damaged or worn products must be disposed of.
- For the best results when processing instruments, instrument stands can be used.

Pre-cleaning on site after use

Note:

- In order to prevent residue from drying on the instruments, they should be cleaned within an hour after use.
- Grinding caps and cap mandrels must be separated every time before cleaning, disinfection, and sterilization.
- In order to prevent the detached grinding caps from falling out of the instrument stands, place the caps on a retaining pin (MC300RS).

Pre-cleaning:

- 1) Wear personal protective equipment (mouth, nose, and eye protection and suitable gloves). Remove all coarse soiling (e.g., blood, tissue, and composites) from the instruments with cellulose swabs after use.
- 2) Then place the instruments in a suitable container with a suitable cleaning agent (Alpro Medical AlproZyme; 0.5% solution for max. 60 minutes). Always follow the instructions of the cleaning agent and disinfectant manufacturer.

Automated cleaning and disinfection

Notes:

- Observe any material incompatibilities (e.g., do not insert instruments that are not made of stainless steel).
- Only use cleaning and disinfection devices that comply with EN ISO 15883.
- Only use a validated cleaning agent with a CE mark (e.g., Dr. Weigert neodisher MediClean advanced).
- Remove any coarse soiling before automated cleaning (see pre-cleaning).

Processing:

- 1) Before cleaning, rinse the instruments under clean, running water for at least 1 minute.
- 2) For optimum cleaning and to prevent damage to the instruments, place them in a suitable instrument stand.
- 3) Place the instruments in the cleaning and disinfection device (CDD) so that all surfaces can be properly rinsed, cleaned, and disinfected.
- 4) Select and start a cleaning and disinfection program with an A_0 of $\geq 3,000$ or at least 5 minutes at 90°C.
- 5) Once the cleaning and disinfection cycle is finished, remove the instruments, check whether they are dry, and, if necessary, dry with medical-grade compressed air.

Alternative cleaning: Manual procedure

Notes:

- Automated cleaning is preferred because it has a higher reproducibility.
- Do not heat the ultrasonic bath above 45°C in order to prevent protein fixation.
- Use only cleaning agents that are approved for cleaning rotating instruments.
- If an instrument stand is not used during ultrasonic cleaning, instruments may come into contact with each other, thereby causing damage to the blades and surfaces.

Cleaning:

- 1) Rinse the instrument under clean, running water for at least 1 minute. Use a soft plastic brush to remove any coarse soiling.
- 2) Instruments made up of several parts (e.g., grinding caps and mandrels) must be disassembled.
- 3) Place the instruments in a suitable instrument stand.
- 4) Clean the instruments according to the instructions provided by the cleaning agent manufacturer in a cleaning solution in an ultrasonic bath. BUSCH recommends AlproZyme 0.5% (Alpro Medical GmbH) for 2 minutes.
- 5) Once the ultrasonic cycle is finished, rinse the instruments under clean, running water for at least 1 minute.
- 6) Dry the instruments with medical-grade compressed air.
- 7) Check that the instrument is clean. If there is residual soiling, the cleaning process must be repeated.

Alternative: Manual procedure

Notes:

- Use only cleaning agents that are approved for disinfecting rotating instruments.
- If an instrument stand is not used during ultrasonic cleaning, instruments may come into contact with each other, thereby causing damage to the blades and surfaces.

Disinfection:

- 1) Place the previously disassembled, cleaned, and checked instruments in a suitable instrument stand.
- 2) Disinfect the instruments according to the manufacturer's instructions in a disinfection solution in an ultrasonic bath. BUSCH recommends BIB forte eco 2% (Alpro Medical GmbH) for 5 minutes.
- 3) Once the ultrasonic cycle is finished, rinse the instruments under clean, running water for at least 1 minute.
- 4) Dry the instruments with medical-grade compressed air.

Reprocessing instructions

Dept.: QM / RA

Date: July 10, 2024

Rev.: 03

Inspection
<p>Notes:</p> <ul style="list-style-type: none">• Use a magnifying glass with 6x magnification.• Cavities must be thoroughly checked because they are difficult to access. <p>Inspection:</p> <ol style="list-style-type: none">1) Perform a visual inspection. Check the instruments for damage, wear, and residual contamination. Heavily worn or damage instruments must be disposed of. Dirty instruments must be re-cleaned.
Packaging
<p>Notes:</p> <ul style="list-style-type: none">• In accordance with KRINKO – 2012, instruments for invasive applications (critical A & B medical devices) must be packaged before sterilisation.• Instruments that are not rustproof must be suitably prepared and protected from rusting (e.g., medical rust protection – B. Braun AESCULAP STERILIT oil drops).• Use only suitable packaging material according to EN ISO 11607-1.• Ensure that the instruments cannot pierce through the packaging. <p>Packaging:</p> <ol style="list-style-type: none">1) Make sure the packaging is large enough in order to prevent tension over the instrument and therefore possible damage.2) Put the instruments in the packaging and seal it.3) Check the packaging. If the packaging is damaged, repackage the instruments.
Sterilization
<p>Notes:</p> <ul style="list-style-type: none">• Unless they were disinfected beforehand, semi-critical A & B medical devices must undergo thermal disinfection in a steam sterilizer. Before thermal disinfection, place the instruments in a suitable instrument stand.• For sterilisation, use only devices that comply with EN 285 or EN 13060. The use of a class B sterilizer (fractionated vacuum process) is validated and recommended.• Instruments that are not rustproof must be suitably prepared and protected from rusting (e.g., medical rust protection – B. Braun AESCULAP STERILIT oil drops). <p>Sterilization:</p> <ol style="list-style-type: none">1) Set the following parameters for pre-vacuum steam sterilization:<ul style="list-style-type: none">○ Min. sterilisation temperature: 132°C○ Min. holding time: 3 minutes○ Min. drying time: 20 minutes2) Place the (packaged) instruments into the autoclave.3) Perform the sterilization cycle using the given parameters.4) Once the sterilization process has been successfully completed, remove the sterilized instruments, and let them cool.5) Check the packaging for damage. If the packaging is damaged, repack and resterilize the instruments.
Storage
<p>Notes:</p> <ul style="list-style-type: none">• Store the sterilized products in a dust-free and dry place in order to ensure that they remain sterile until use.
Service life and disposal
<p>Notes:</p> <ul style="list-style-type: none">• The service life of rotating instruments depends on their use and the associated wear and tear. There is no limit to the number of processing cycles permitted (except for disposable products). The instruments must be checked after each use (see Inspection) and replaced where necessary.• The instruments can be disposed of in normal household/practice waste.

These processing instructions comply with DIN EN ISO 17664:2021 and the recommendation of the Robert Koch Institute (KRINKO 2012)

The latest version of these instructions is always available at www.busch.eu.

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